



BIOSYNERGY, INC.

1940 E. Devon Avenue, Elk Grove Village, Illinois 60007
(847) 956-0471 Toll Free (800) 255-5274 Fax (847) 956-6050
e-mail: biosynergy@att.net website: www.biosynergyinc.com

TEMPTREND® INDICATORS FOR DRUG TESTING VALIDATION

TempTrend® indicators are disposable liquid crystal surface temperature indicators with a temperature range of 87°F to 103°F, ideal for measuring the temperature of urine specimens for drug testing purposes. The TempTrend® indicator is about the size of a postage stamp, self-adhering, easy to use and gives a temperature reading at a glance.

To use the TempTrend® indicator, simply peel off the backing and attach the indicator directly onto the specimen container and within seconds the numbers will light up verifying the temperature of the urine specimen. A freshly voided urine specimen should approximate body temperature. Irregular temperatures may be evidence of altered urine specimens. TempTrend® indicators will indicate whether or not the urine specimen is within the freshly voided temperature range and can therefore be used to verify sample validity.

The TempTrend® indicator is a single use monitor and can be attached to all types of urine specimen containers. It produces a single-spot reading in 1-15 seconds providing a fast, easy method of checking urine specimen temperatures. The TempTrend® indicator can be purchased by the box or in bulk quantities. For additional information about the TempTrend® indicator or a **free sample**, simply call us **toll-free at 800-255-5274** or email biosynergy@att.net.